
AESCULAP® VATS INSTRUMENTS

VIDEO-ASSISTED THORACOSCOPIC SURGERY

DISASSEMBLY


1
Release the locking mechanism on ratchet handle
Push the actuating lever forward.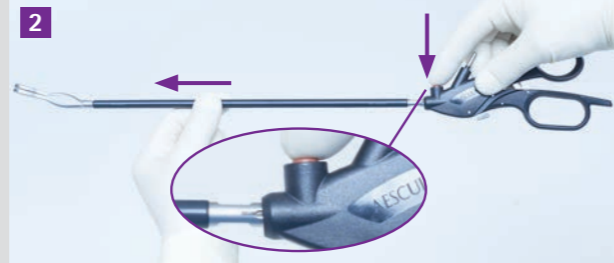


2
Disconnect shaft and handle
Press the red button, remove the shaft by pulling the instrument shaft out of the handle.
IMPORTANT: Do not hold on the movable part of the handle (lower handle component).


3
Disassemble shaft I
Hold the jaw insert at the tip and remove the outer insulation tube.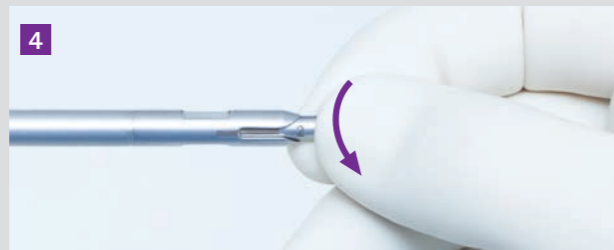


4
Disassemble shaft II
Rotate the push rod 90° counterclockwise.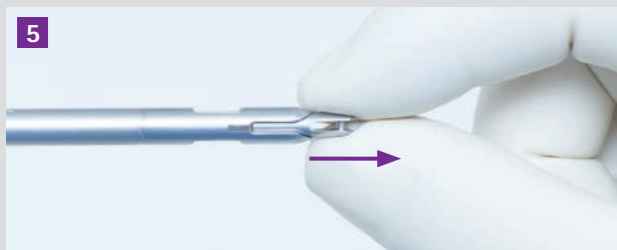

5
Disassemble shaft III
Remove the push rod.

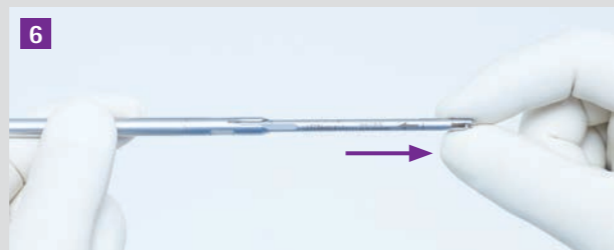
6
Disassemble shaft IV
Slide the black outer tube as far as it will go over the jaw insert.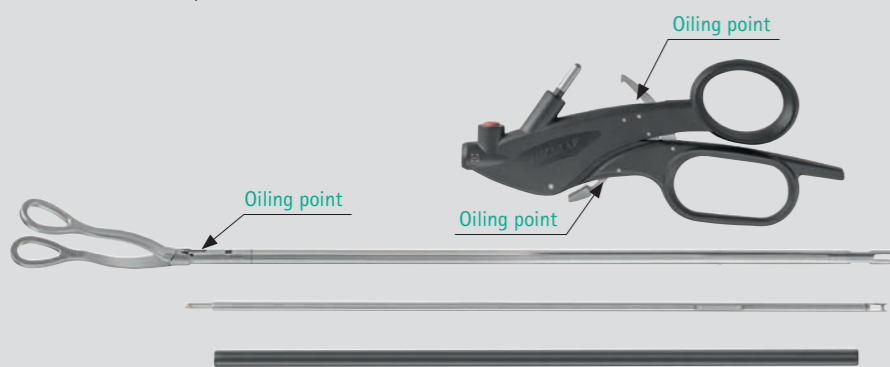


JG600
including 2 spray nozzles
Aesculap Oil spray in
spray cans (CFC-free)
for instrument care prior
to sterilization


ASSEMBLY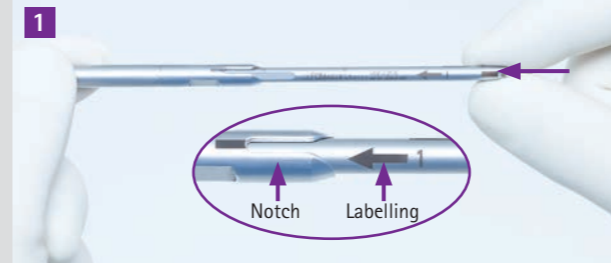


1
Assemble shaft I
Slide the push rod into the jaw insert as far as it will go. Making sure that the labelling on the push rod (←1) is in line with the notch of the jaw part.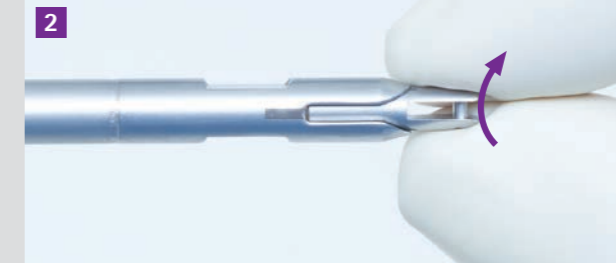


2
Assemble shaft II
Rotate the push rod 90° clockwise until it locks into place.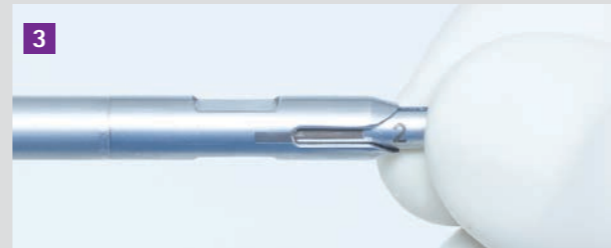


3
Assemble shaft III
The labelling on the push rod (←2) is in line with the notch.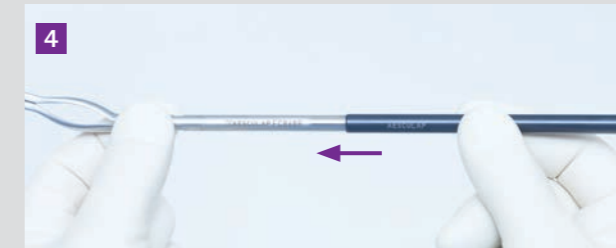

4
Assemble shaft IV
Slide the black outer tube as far as it will go over the jaw insert.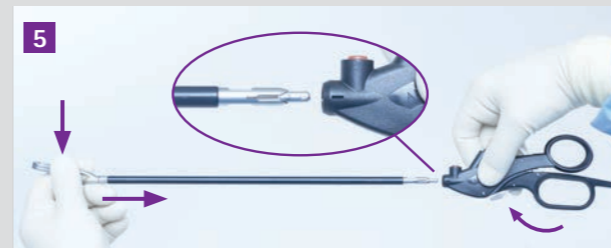


5
Connect shaft and handle I
Release the locking mechanism on the ratchet handle by pushing the actuating lever forward. Hold the tip of the shaft (do not hold on the black outer tube). With the other hand, hold the handle at the labelling.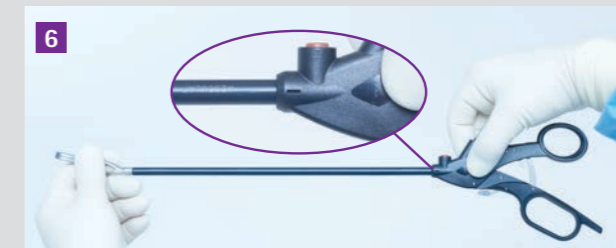

6
Connect shaft and handle II
Slide the shaft into the handle until it locks into place. Observe the markings. The moveable part of the handle will move down.

IMPORTANT: Do not touch the moveable part of the handle (lower handle component).

Test the assembly of the instrument by opening and closing the handle to ensure that jaw is opening and closing.

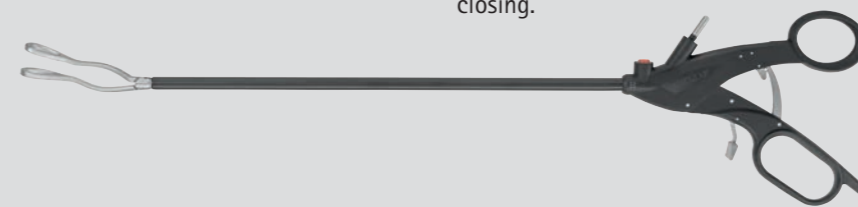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