



**SURFACE
DISINFECTION**



Meliseptol[®] acute

FAST AND HIGHLY EFFECTIVE

Meliseptol[®] acute

FAST AND HIGHLY EFFECTIVE

Fast acting alcohol based disinfectant for small patient near surfaces and medical devices – effective against bacteria, yeasts, mycobacteria, enveloped and non-enveloped viruses within a minute.

PROPERTIES

- Solution for rapid disinfection of non-invasive medical devices and small areas
- Ready-to-use alcohol based disinfectant
- Broad efficacy: full virucidal activity
- Good material compatibility, e.g. examination chairs, alcohol resistant surfaces
- Perfume-free, mild smell
- Compatible with B. Braun Wipes dispenser system

IN A NUTSHELL

- Fast and effective for alcohol resistant material
- Short exposure time
- Free from quaternary ammonium compounds (QAC)
- Aldehyde-free
- Alkylamine-free
- Colourless
- Available as ready-to-use Wipes in Flowpacks
- Listing: VAH list and the IHO List of Disinfectants

APPLICATION INSTRUCTIONS

Soak the areas fully with Meliseptol[®] acute and rub with a wipe or use the ready-to-use Meliseptol[®] acute wipe. Keep the surface wet until the exposure time has elapsed.

Do not wipe the surface dry. For exposure time, see table.

Physico-chemical data ready-to-use solution

Appearance	clear, colorless liquid
pH-value	ca. 2-3
Density (20°C)	ca. 0.94 g/ml
Fragrance	none
Flash point (DIN 51 755)	+ 33.5°C

EXPOSURE TIMES

Spectrum of activity Use undiluted under clean conditions at ambient temperature (min. 20°C)	Test Norms	Exposure Time
Bactericidal and yeasticidal incl. C. auris with mechanical action	EN 16615, EN 13727, EN 13624	1 min.
Bactericidal and yeasticidal incl. C. auris without mechanical action***	EN 17387**, EN13727, EN 13624	1 min.
Mycobactericidal with mechanical action	EN 14348, EN 16615	1 min.
Fungicidal (A. brasiliensis) without mechanical action***	EN 17387**, EN 13624	30 min.
Virucidal, incl. HBV, HCV, HIV*	EN 14476	1 min.
Virucidal without mechanical action***	EN 14476, EN 16777	3 min.
Adenovirus	EN 14476	1 min.
Norovirus, MNV	EN 14476	1 min.
Poliovirus	EN 14476	1 min.
Polyomavirus SV40	EN 14476	1 min.

* Acc. recommendation of RKI, Bundesgesundheitsbl. 03-2017

** Some tests were carried out acc. modified EN 13697/VAH 14.1 with conditions and acceptance criteria of EN 17387

*** Additional information: main application will be with mechanical action



PRODUCT	REF
250 ml spray bottle	180241, 180246
250 ml round bottle	180242, 180247
750 ml spray bottle	180243, 180248
1000 ml OV bottle	180244, 180249
5 l canister	180245, 180250
100 pcs flow pack 100, 18 cm x 20 cm	180255, 180296
42 pcs flow pack XL, 24 cm x 30 cm	180256, 180297

Meliseptol[®] acute – composition:

100 g solution contains: 39g Propan-1-ol, 0.98g Glycolic acid, Excipients