Askina® Calgitrol® Ag

en Silver alginate wound dressing

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es Apósito de alginato de plata

fr Pansement Alginate-Argent

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en INSTRUCTIONS FOR USE

1. PRODUCT DESCRIPTION

Askina® Calgitrol® Ag is a sterile dressing consisting of two layers:

• an absorbent polyurethane foam layer which provides for the absorption of wound exudate

• an ionic silver alginate matrix which provides for broad antimicrobial effectiveness and helps prevent contamination from external bacteria; in the presence of wound exudate, the silver alginate matrix helps maintain a moist wound environment conducive to natural healing conditions.

2. INDICATIONS

Askina® Calgitrol® Ag is indicated for the management of exuding partial to full thickness wounds, stage I-IV pressure sores, venous ulcers, second degree burns and donor sites. It is indicated for external use only and may be used in the management of infected wounds under medical supervision at the discretion of the physician.

PRECAUTIONS

In the event of clinical infection, topical silver does not replace the need for systemic therapy or other adequate infection treatment.

This medical device contains 4.81mg of propylene glycol per cm² of dressing, which is equivalent to 0.069 mg/kg body weight/ cm².

Propylene glycol may cause skin irritation. Do not use this medicine in babies less than 4 weeks old with open wounds or large areas of broken or damaged skin (such as sunburn) without talking to your doctor or pharmacist.

Warning: Re-use of single-use devices creates a potential risk to the patient or user. It may lead to contamination and/ or impairment of functional capability. Contamination and/or limited functionality may lead to injury, illness or death of the patient. Sterile used packs damaged or opened.

4. CONTRAINDICATIONS

Askina® Calgitrol® Ag should not be used on:

- Patients with a known sensitivity to alginates or silver.
- Askina® Calgitrol® Ag should not be used:
 - Where the presence of metals is contraindicated.
 - Askina® Calgitrol® Ag is contraindicated for use on patients with certain infections under conditions, such as:
 - Tuberculosis
 - Syphilis
 - Deep fungal infection
 - Thrice degree burns.

5. INFORMATION FOR USE

Askina® Calgitrol® Ag is very simple to apply and requires no special skills or equipment. Askina® Calgitrol® Ag may be left in place for up to seven days or until the dressing is saturated with wound exudate.

5.1 Skin Preparation

Clean the skin with sterile saline (0.9%) or an appropriate wound cleansing solution such as Prontosan® and sterile swabs. Dry the skin surrounding the wound.

5.2 Dressing Application

Select the appropriate Askina® Calgitrol® Ag size that will completely cover the wound surface, ensuring a 2 to 3cm margin beyond the edges of the wound. If necessary, several dressings can be overlapped to cover very large wound areas. Place the dressing with silver matrix (dark surface) facing the wound. Ensure Askina® Calgitrol® Ag with a secondary dressing or secure with surgical tape or bandage as appropriate.

5.3 Dressing Removal/Changes

Remove retention dressing if applicable. Gently lift Askina® Calgitrol® Ag from the wound. If the dressing adheres to the wound gently irrigate with saline or Ringer solution. Follow procedures 5.1 and 5.2 to apply a new dressing.

6. SPECIAL NOTES

The wound may initially appear to increase in size as the early stages of Askina® Calgitrol® Ag treatment. This is normal and occurs as any wound debris is removed from the edges of the wound. This clears the way for healing. In the management of moderately exuding wounds, Askina® Calgitrol® Ag can only make the overlying environment more conducive to healing. There are times when healing is impaired as a result of underlying conditions; in these instances, Askina® Calgitrol® Ag alone may make little or no progress, and suitable treatment of the underlying conditions will be necessary as well. Therefore, if after 4 weeks of Askina® Calgitrol® Ag treatment there has been no improvement then, in line with accepted wound management practice, the original diagnosis and overall therapy should be reassessed.

Askina® Calgitrol® Ag should be left in place as long as possible in order to prevent contamination of the fragile newly formed tissue and to reduce cross contamination through frequent dressing changes. Thick necroses should be removed before applying Askina® Calgitrol® Ag. Clinicians/ Healthcare Professionals should be aware that there is very limited data on prolonged and repeated use of silver containing products and particularly in children and neonates. At present there are no contraindications for the pediatric use of silver. To avoid inappropriate treatment, it is recommended that official best practice guidance for wound management be followed.

7. PRESENTATION

Askina® Calgitrol® Ag is available in the following sizes:

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