



ASKINA® FOAM RANGE

HYDROPHILIC FOAM DRESSINGS

B. Braun Wound Care. Right. From the start.

Askina® Foam | Askina® Foam Cavity | Askina® Trachea

HYDROPHILIC FOAM DRESSINGS



ADVANTAGES

- Excellent fluid handling characteristics
- Enhances a moist wound environment
- No adherence to the wound: designed to allow atraumatic, painless dressing changes
- Askina® Foam Cavity is highly resistant: easy to remove, without leaving particles

Askina® Foam is a two layered non-adherent foam dressing consisting of


- A soft hydrophilic polyurethane breathable foam layer with high fluid handling capacity
- A thin, transparent and protective polyurethane film, waterproof and bacteria resistant

Askina® Foam Cavity is made of the same foam material as Askina® Foam, without the polyurethane backing. The specific shape, absorption capacity and resistance makes it ideal for dressing deep, highly exuding wounds.

Askina® Trachea is a hydrophilic foam dressing with the circular aperture designed to fit neatly around a tracheostomy tube or other drain or stoma.

TIPS

- Askina® Foam can be fixed with tapes or bandages, or with compression bandages
- Askina® Foam can also be used as a secondary dressing for exuding wounds, in combination with Askina® Sorb, or as a secondary dressing for deep wounds, filled with Askina® Sorb Rope, or Askina® Foam Cavity

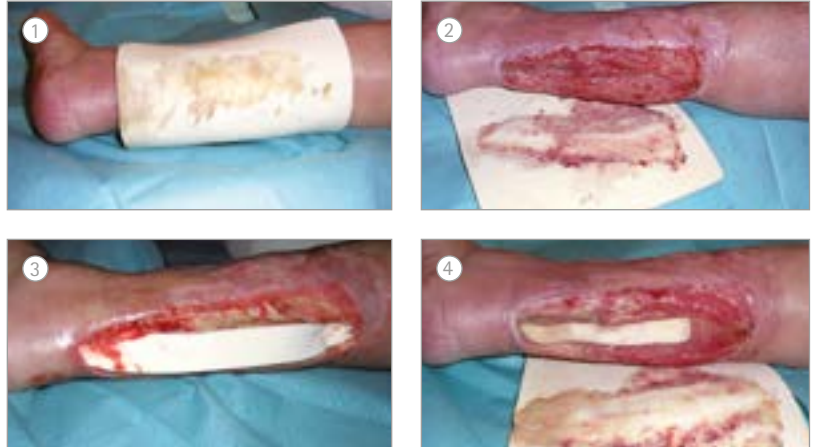

ASKINA® FOAM INDICATIONS

Moderately to heavily exuding wounds:

- Pressure ulcers
- Venous and arterial leg ulcers
- Diabetic foot ulcers
- 1st and 2nd degree burns

ASKINA® FOAM CAVITY INDICATIONS

- Deep cavity wounds
- Sinuses
- Wound tunnels

ASKINA® TRACHEA INDICATIONS

- Absorbent apertured foam dressing for the stoma created by tracheostomy or for wound drainage sites.

ASKINA® FOAM: EXAMPLE OF USE

- ① Askina® Foam used on exuding flat wound.
- ② Vertical absorption, without maceration of the wound edges.
- ③ ④ Combined use of Askina® Foam Cavity, used to fill a deep exuding wound, and Askina® Foam, used as an additional absorption dressing.



Askina® Foam	Size	Pcs/Pack	REF
Askina® Foam	5 cm x 7 cm	10	7240710
	10 cm x 10 cm	10	7241010
	10 cm x 20 cm	10	7241210
	20 cm x 20 cm	5	7242005
Askina® Foam Cavity	2.5 cm x 40 cm	10	7244010
Askina® Trachea	9.5 cm x 8.5 cm	10	7248510

Manufactured by:

B. Braun Hospicare Lt. | Collooney | Co. Sligo | IRELAND
www.bbraun.com/wound-management

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and/or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun. Non-binding documents and photographs. For healthcare professional use only.