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1. PRODUKTBESCHREIBUNG

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GB INSTRUCTIONS FOR USE

1. PRODUCT DESCRIPTION

Askina® Trachea is a sterile, hydrophilic foam dressing that includes a polyurethane foam wound contact surface with a high absorption capacity and a vapour permeable, water and bacteria resistant polyurethane film outer layer.

The circular aperture in the dressing is designed to fit neatly around a tracheostomy tube or other drain or stoma.

2. INDICATIONS

Absorbent apertured foam dressing for the stoma created by tracheostomy or for wound drainage sites.

3. CONTRAINDICATIONS / SAFETY INFORMATION

In case of infection with inflammatory signs, (temperature, oedema, redness, pain) contact a proper medical authority. Resume use of Askina® Trachea when normal healing conditions are present again.

Askina® Trachea dressing should not be reused.

4. INFORMATION FOR USE

Do not use if pack is damaged or opened.

Askina® Trachea is simple to apply, requiring no special skills. Daily changes may be required at the beginning of treatment but the dressing may be left in place undisturbed for up to 7 days, or until the dressing is saturated.

4.1 Skin Preparation

- Cleanse and dry the area surrounding the tracheostomy tube in accordance with normal procedures.

4.2 Dressing Application

- With the "slit" from the aperture positioned above the tube, fit Askina® Trachea around the tracheostomy tube with the foam side (side with no print) directly to the wound surface and fix it with an appropriate secondary dressing e.g. a fixation dressing such as Askina® Derm may be use.

4.3 Dressing changes

- Askina®Trachea should be changed when the dressing is saturated with exudate. The dressing may be left in place up to 7 days when there is little exudate. Where

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